



Installation Instructions for the RC5300W and RC5300R AED Cabinet

RC5300W (Surface Mount) Cabinet Mounting:

- 1) Locate studs or identify structural material capable of providing sufficient support.
- 2) Select fastener type (**not included**):
 - a) Steel studs—#10 x 2 1/2" fine thread
 - b) Wood studs—#10 x 2 1/2" coarse thread
 - c) Block or concrete—plastic wall anchors of minimum 1 1/2" depth
 - d) Hollow wall—minimum grip of 5/8"
- 3) There are 4 mounting holes in the cabinet back intended for cabinet mounting. It is recommended that fasteners are utilized in all 4 holes. Position the cabinet such that a minimum of 2 fasteners attach to studs or concrete.
- 4) Fasten the cabinet to the wall.
- 5) Ensure door opens freely.

RC5300R (Recessed Mount) Cabinet Mounting:

- 1) Prepare a rough opening of 17 1/4" high by 16 3/8" wide.
- 2) Place the cabinet inside the rough opening and level.
- 3) There are 2 mounting holes on each cabinet side intended for cabinet mounting. Secure with four #10 screws. (**not included**)
- 4) Ensure door opens freely.

Activate the Alarm:

- 1) Insert the key into the activation switch mounted in the cabinet .
- 2) Turn the key to the unarmed position as shown in Figure 1.
- 3) Install the 9 volt battery into the holder mounted on the under side of the cabinet top.
- 4) Connect the 9 volt battery clip to the battery.
- 5) Close the cabinet door.
- 6) Turn the key to the armed position as shown in Figure 1.
- 7) Opening the door will trigger the alarm. Closing the door will stop the alarm.

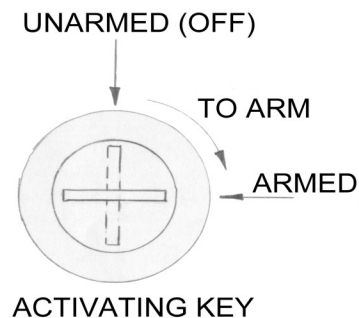


Figure 1

Maintenance:

Replace the battery yearly or as needed.

Warranty:

One year limited warranty.

*HeartStation
1000 Brown Street, Wauconda, IL 60084
(800) 818-2939*